Parcus Non-cannulated Headed Reamers

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0086
EC REP

Parcus Non-cannulated Headed Reamers (English)

1. Indications

Parcus Non-cannulated Headed Reamers are designed to be used to create sockets and tunnels in bone.

2. Warnings

- A. This product is intended for use by or on the order of a physician.
- B. This product is supplied sterile. Do not use this product if the packaging has become wet or damaged in any way.
- C. This product is designed for Single Use Only and should not be re-sterilized.
- D. Never use a bent or damaged reamer.
- E. This product is not intended to be left in the body.

3. Material Specifications

This product is made from a stainless steel alloy. The material used in the manufacture of this device is radio-opaque, and can be detected with conventional X-ray or Fluoroscopy.

4. Packaging and Labeling

- A. Check the sterile packaging for damage of any kind.
- B. Do not use the product if the sterile packaging has been compromised in any way.

5. Sterilization

This product is supplied in sterile packaging. The contents are sterilized by EO gas. These products must never be re-sterilized.

6. Storage

Products must be stored in the original unopened package in a dry place and must not be used beyond the expiration date indicated on the package.

7. Directions for Use

- A. Select and verify a Non-cannulated Reamer of appropriate diameter.
- B. Open the product using aseptic technique.
- C. Place the non-fluted end of the reamer in a purpose designed drill motor or reamer.
- D. Place the tip of the Non-Cannulated Reamer at the site for the creation of the tunnel or socket.
- E. Prudently advance the Non-cannulated Reamer into the bone to the desired depth, using the drill motor or reamer set to rotate in a clockwise direction.
- F. Withdraw the Non-cannulated Reamer from the bone, remove it from the chuck of the drill motor or reamer and dispose it in an appropriate manner.

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